VacuMax™

Portable Aspirator

605

Patient Manual

CAUTION: USA Federal law restricts this device to sale by or on the order of a physician.

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Specifications	2
Important Parts of the VacuMax	3
Setup and Operating Instructions	4
Battery Charging and Maintenance	9
Cleaning	10
Maintenance	11
Troubleshooting	13
Important Safety Information	14
Warranty	15

Rx ONLY
The portable VacuMax weighs only 7 lbs and comes complete with its own carry bag. This unit operates on AC, DC or internal battery, making it ideal for patient mobility or as back-up to a larger AC only unit.

SIZE	10" L x 7" W x 8" H
SIZE	25 cm x 18 cm x 20 cm
WEIGHT	7 lbs. 3 oz. (3.26 kg)
ELECTRICAL REQUIREMENTS:	120V AC 60Hz or 12V DC internal battery adapter
ELECTRICAL REQUIREMENTS:	120V AC 60Hz or 12V DC internal battery adapter
ELECTRICAL REQUIREMENTS:	230V AC 50/60Hz or 12V DC internal battery adapter
OPERATING TEMPERATURE	Ambient conditions (min 50°F to max 104°F) (min 10°C to max 40°C)
STORAGE CONDITIONS	Room temperature preferred (min -4°F to max 140°F) (min -20°C to max 60°C)
BATTERY SPECIFICATION	12V DC, 2000mAh, 6 cell sealed lead acid

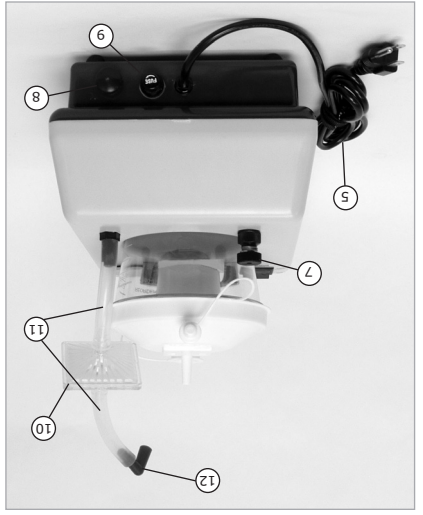
Important Parts of the VacuMax System

1. Suction Jar Inlet
2. Gauge
3. On/Off Switch
4. DC Power Jack
5. AC Power Cord
6. Connecting Tube
7. Suction Regulator Knob
8. Filter Cap
9. Fuse
10. Hydrophobic Filter
11. (2) Suction Tube(s)
12. Elbow
13. 12V DC Adapter Automobile
14. Non-Conductive Tubing (Not Shown)

FRONT VIEW

1. Suction Jar Inlet
2. Gauge
3. On/Off Switch
4. DC Power Jack
5. AC Power Cord
6. Connecting Tube
7. Suction Regulator Knob
8. Filter Cap
9. Fuse
10. Hydrophobic Filter
11. (2) Suction Tube(s)
12. Elbow
13. 12V DC Adapter Automobile
14. Non-Conductive Tubing (Not Shown)

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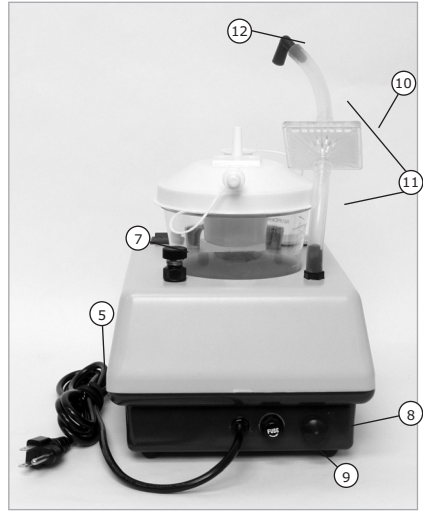
POWER OPTIONS

You have either purchased a 110V AC only, or a 230V AC only unit. These units are not interchangeable. Please refer to the serial number tag on the bottom of the unit for unit identification. Please read instructions in this manual regarding operation for these different types of devices.

AT HOME (Using AC Power)

1. Make sure the power cord is unplugged and the power switch is in the OFF position.
2. Inspect the unit, including the power cord, for any damage. Contact Care Provider to have any damaged parts replaced.
3. Set the VacuMax on a level, sturdy surface so the controls of the unit can be easily reached and adjusted.
4. Connect suction tubing, elbow and filter assembly to the outlet on the regulator and inlet on the collection jar as shown on page 3. Make sure the tubing from the collection jar is connected to the 'Fluid Side' of the hydrophobic filter. Make sure all tubing connections and the collection jar lid are pushed on tightly to ensure a proper vacuum level is achieved.
5. Connect the AC power cord located on the back of the unit to a standard 110V AC or 230V AC wall outlet (voltage depending on type of unit you own.)

CAUTION

Never submerge the VacuMax in water, hold under running water or spray with water or other liquids. This will result in compressor damage and could result in electrical shock.

WARNING

To avoid the risk of possible electrical shock, VacuMax must be operated on the specified power source.

NOTE
The in-line hydrophobic filter is designed to prevent foreign material from being drawn into the compressor. At a minimum, change the filter whenever foreign material is drawn into the tubing.

NOTE
Operating the VacuMax device without the hydrophobic filter can result in damage to the internal components of the device and will void the warranty. The hydrophobic filter should be replaced monthly and/or between clients, whichever is sooner.

NOTE
Do not move VacuMax with a full collection jar. Liquids may flow back through the device resulting in damage to the internal components of the compressor.

(Continued)

6. Turn the switch to the ON position. The red LED on the switch will turn on.
7. Set suction level by simultaneously occluding the outlet of the collection jar and turning the regulator knob until the required suction registers on the gauge. Turn the regulator clockwise to increase vacuum and counter-clockwise to decrease vacuum. Observe the vacuum level on the unit's gauge.
8. Attach the non conductive connecting tubing to the outlet on the suction jar.
9. Attach your catheter (not included) to the connecting tubing.
10. Proceed with the treatment.
11. At the conclusion of the treatment, turn the switch to the OFF position.

AWAY FROM HOME

(Using internal/external DC power)

External DC Power

1. Make sure the DC power cord is unplugged and the power switch is in the OFF position.
2. Inspect the unit, including the DC power cord and DC Connector, for any damage. Contact Care Provider to have any damaged parts replaced.
3. Set the VacuMax on a level, sturdy surface so the controls of the unit can be easily reached and adjusted.
4. Connect suction tubing, elbow and filter assembly to the outlet on the regulator

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5. Connect the DC power cord to the DC jack located on the front of the unit. Make sure the cord is pushed in all of the way for a solid connection.
6. Connect the auto adapter plug of the DC cord to the auto adapter receptacle in your vehicle.
7. Turn the switch to the ON position. The red LED on the switch will turn on.
8. Set suction level by simultaneously occluding the outlet of the collection jar and turning the regulator knob until the required suction registers on the gauge. Turn the regulator clockwise to increase vacuum and counter-clockwise to decrease vacuum. Observe the vacuum level on the unit's gauge.
9. Attach the non conductive connecting tubing to the outlet on the suction jar.
10. Attach your catheter (not included) to the connecting tubing.
11. Proceed with the treatment.
12. At the conclusion of the treatment, turn the switch to the OFF position.

NOTE
Cold temperatures may result in poor compressor performance. Bring machine up to room temperature for best results.

NOTE
A flashing switch light during battery operation indicates a low battery and device performance may drop quickly. Refer to the Battery Charging section for more information.

Internal DC Power: Battery

1. Make sure the power switch is in the OFF position.
2. Inspect the unit for any damage. Contact Care Provider to have any damaged parts replaced.
3. Set the VacuMax on a level, sturdy surface so the controls of the unit can be easily reached and adjusted.
4. Connect suction tubing, elbow and filter assembly to the outlet on the regulator and inlet on the collection jar as shown on page 3. Make sure the tubing from the collection jar is connected to the 'Fluid Side' of the hydrophobic filter. Make sure all tubing connections and the collection jar lid are pushed on tight to ensure a proper vacuum level is achieved.
5. To use this option, simply turn the device switch to the ON position while the device is not plugged into any other power source. The red LED on the power switch will turn on. If the red LED on the power switch is flashing, that indicates that the battery is low and needs to be recharged. See Battery Charging Section for more information.

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6. Set suction level by simultaneously occluding the outlet of the collection jar and turning the regulator knob until the required suction registers on the gauge. Turn the regulator clockwise to increase vacuum and counter-clockwise to decrease vacuum. Observe the vacuum level on the unit's gauge.
7. Attach the non conductive connecting tubing to the outlet on the suction jar.
8. Attach your catheter (not included) to the connecting tubing.
9. Proceed with the treatment.
10. At the conclusion of the treatment, turn the switch to the OFF position.

(Continued)

To properly charge the internal battery, giving it the longest life possible, please read and follow the directions below.

1. To charge the battery, connect the unit to an external power source (either a 110V AC, a 230V AC or a 12V DC external power source). Set the power switch to the OFF position. The OFF position is also the charging position. The power switch will not light while the battery is charging.

2. To fully charge the battery, eight hours of charging is necessary. However, if the user does not have time to fully charge the battery, the unit should still be charged as long and as often as time permits. Full discharge will shorten battery life. **The battery will not be damaged by overcharging under normal use.**

3. It is not recommended for the battery to stay discharged or unused for a long period of time (60 days or more). **Charge the battery at least once every two months.**

POWER SWITCH CONSIDERATIONS

While in use, a flashing power switch alerts the user that the battery is exhausted and another power source must be used in place of the battery. A new, fully charged battery should last about 45 minutes.

NOTE

Charging the battery in your vehicle without the engine running may drain the vehicle's battery.

NOTE

It is normal for rechargeable batteries to lose capacity and useful life over time. Life expectancy of battery will depend on number of uses and proper maintenance.

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1. Make sure the ON/OFF switch is in the OFF position.
2. Unplug the aspirator from the wall outlet.
3. Wipe the aspirator with a clean, damp cloth every 2-3 days. This will keep the unit dirt and dust free.

TUBING AND COLLECTION JAR

1. Verify the device is turned OFF and allow the suction to drop off.
2. Disconnect the tubing from the collection jar and from the device.
3. Rinse the tubing and collection jar thoroughly with three parts hot tap water and one part vinegar.
4. Rinse the parts with hot tap water and allow them to air dry.
5. Reconnect the tubing and hydrophobic filter between the collection jar and the device.

WARNING

Do not remove the

compressor case/cabinet.

Failure to comply could

result in electrical shock. All

disassembly must be

performed by an authorized

evo service center.

CAUTION

Never submerge the

VacuMax in water, hold

under running water or

spray with water or other

liquids. This will result in

compressor damage and

shock.

All maintenance on your VacuMax system must be performed by an authorized EVO service center.

UNIT AIR FILTER CHANGE

VacuMax contains an external air filter located on the back of the device. This filter requires periodic replacing to ensure the proper function of the device. It is recommended that this filter be changed monthly or sooner, depending on the cleanliness of the operating environment. To replace the filter, follow the steps below:

1. Remove the filter cap by unscrewing counterclockwise.
2. Remove the filter from the cap and replace it with a new one.
3. Screw the cap back on.

NOTE
Maintenance manual is available through EVO upon request.

NOTE
Should fluid be aspirated back into the unit, dealer service is necessary as possible compressor damage may result.

NOTE
Operating the VacuMax without the hydrophobic filter may allow fluids to enter the compressor causing damage to its internal components.

WARNING
To reduce the risk of electric shock, always unplug the power cord from the power source before servicing.

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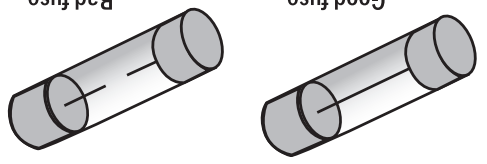
12 VacuMax

A 2A fuse is located on the back of the device. It protects the device from power surges that can occur in the home and from exposing this device to a wrong power source. To replace the fuse, unscrew the fuse holder on the back of the device. Take out the old fuse, insert the new fuse and screw the holder back on.

FUSE CHANGE

1. Disconnect the tubing from both ends of the filter. Discard the old filter.
2. Attach the new filter so the tubing from the suction jar is connected to the 'fluid side' of the filter. Connect the tubing from the pump to the opposite side of the filter.

HYDROPHOBIC FILTER REPLACEMENT

The suction tubing, collection jar and hydrophobic filter supplied with the VacuMax are recommended for single patient use and should be replaced at a minimum between patients and as per physician order. If the filter gets wet, it must be replaced.

SUCTION TUBING/COLLECTION JAR/HYDROPHOBIC FILTER

This EVO product is an electrical device. Always follow basic safety precautions when using this device, including but not limited to the following:

To reduce the risk of burns, fire, electrocution, or other bodily harm or injury to persons:

1. Always unplug this device after each use.
2. Never submerge this product in water.
3. Never use this product while bathing, showering, washing dishes, or around a water source of any kind.
4. Never leave this product unattended.
5. Use this product only for its intended use as described in the operating manual.

6. Closely monitor and supervise use by invalids and children.
7. Never block the air vent openings or allow objects to fall or to be inserted into the air vent openings.
8. Observe caution when using this product where oxygen is being administered.
9. If this product is being used in a life threatening situation, a backup unit is necessary.

WARNING

This equipment is not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.

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There is no other express warranty, implied warranties, including those of merchantability and fitness for a particular purpose, are limited to the duration of the express limited warranty and to the extent permitted by law and all implied warranties are excluded. This warranty does not cover providing a loaner compressor, compensating for costs incurred for compressor rental, or labor costs incurred in repairing or replacing defective part(s).

NOTE

eVO warrants the VacuMax portable aspirator to be free from defective workmanship and materials for a period of 1 year from the date of purchase. The internal battery shall be warranted for a period of 3 months from the date of purchase. Any defective part or assembly will be repaired or replaced, at the sole discretion and determination of eVO if the unit has not been misused or tampered with during the warranty period. Normal maintenance items, as outlined in this manual and disposable components are not covered by this warranty. Shipping charges, if any, shall be paid by the purchaser.

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EVO Medical Solutions
2636 289th Place
Adel, IA 50003
515.993.5001
515.993.4172 f
800.759.3038 (toll free)