

Replacement Parts & Accessories

PART	REORDER NUMBER
Medication Cup Cap	22210
MicroPump with Chamber, Mouthpiece, Reservoir and Cap	22218
Nebulizer Body	22207
Mouthpiece	22205
AC Powered Controller*	21099
Battery Powered Controller with Cable	21105
Battery Powered Controller Cable	22201
Mask Connector Elbow (US)	22203
Carrying Case	29049
Instruction Manual*	22263

To order replacement parts or accessories, visit your local home medical equipment dealer or pharmacy.

Your AeronebGo was tested and inspected at the factory and is a premium device that has a life expectancy of at least one year (730 treatments) with proper care and cleaning. Lack of proper care and cleaning, disassembly of the retaining ring or misuse will void the warranty.

*Contact your local supplier for reorder number specific for your geographical region.

R_x ONLY




Aeroneb[®]Go
Micropump Nebulizer

Quick Start Card

ATTENTION: Please read important cleaning instructions within this quick start card.

Retain this card with your device for future reference.

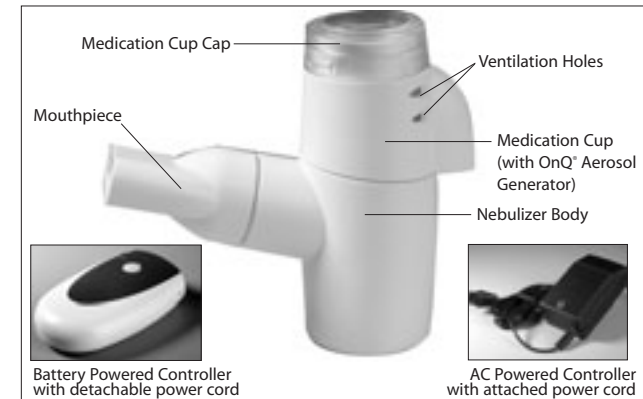

evo evo Medical Solutions
26378 289th Place
Adel, IA 50003
515.993.5001
515.993.4172 f
800.759.3038 (toll free)

Aerogen and Aeroneb are registered trademarks of Aerogen, Ireland Ltd.
©2007 evo Medical Solutions
P/N 22262 Rev B Rx Only



Aerogen (Ireland) Limited,
Galway Business Park,
Dangan,
Galway,
www.aerogen.com



IMPORTANT CAUTION

As with any mechanical device, this product may become unusable due to an electrical outage, battery depletion, or mechanical failure. We recommend that you have spare batteries and have a backup device available to you.

If this product is being used to treat a life threatening condition, a backup device is necessary.

Setup & Use Instructions



Before you begin, run the Medication Cup under hot water, then tap the generator slightly.



STEP 1

Connect the Medication Cup (containing the OnQ[®] Aerosol Generator) to the Nebulizer Body. **NOTE:** Make sure the Aeroneb Go is unplugged before you add medication to the Medication Cup.



STEP 2

Fill the Medication Cup with the prescribed nebulizer medication. Never pour more than 6 mL into the cup. The continuous line on the inside surface of the cup indicates the maximum filling level. (When the medication cup contains medication, the cup should remain in an upright position). **CAUTION:** To avoid damage to the nebulizer unit, do not use a syringe with needle.



STEP 3

Securely place Cap on Medication Cup.



STEP 4

Attach Mouthpiece or Mask Connector and mask to Nebulizer Body. **NOTE:** Masks are an optional accessory and may be purchased through evo Medical Solutions or your Aeroneb[®] Go dealer.



STEP 5

Insert the nebulizer cable connector into the back of the Nebulizer Unit. Slide the cable straight up the nebulizer body.



STEP 6

Plug the AC Powered Controller into a standard AC electrical outlet using the appropriate outlet adapter. The green light on the AC Powered Controller will light up.



OR

Connect the nebulizer cable to the Aeroneb[®] Go Battery Powered Controller.



Turn on the unit by pressing the "on/off" button on the Battery Powered Controller. The green light on the Battery Powered Controller will light up. Visually verify that aerosol is coming out of the mouthpiece or mask to indicate that the device is functioning properly.



NOTE: (1) If you are using batteries and the green light is flashing, your batteries are low. You may finish your treatment, and then replace the three "AA" batteries. **(2)** If the green light only comes on while depressing the button, immediately replace the three "AA" batteries.



STEP 7

Place the mouthpiece between your teeth, with your lips firmly sealed around the mouthpiece. If you use an aerosol mask, place the mask over your mouth and nose. Breathe in and out slowly through your mouth until aerosol formation stops. Do not cover ventilation holes.

NOTE: Aerosol mask is not provided.



OR



STEP 8

Unplug AC Powered Controller or turn off Battery Powered Controller. **Disconnect cable from Nebulizer Unit by sliding the cable straight down the handset, disassemble Nebulizer Unit and rinse Nebulizer Unit Parts after each use.**

Do NOT remove the retaining ring under the medication cup!

NOTE: Removal and the improper insertion of the retaining ring from the underside of the medication cup may cause the unit to leak or malfunction.

Wipe dry the connector end of the cable after each use. Clean the nebulizer unit and associated parts daily.

Please read instructions below or refer to your Aeroneb[®] Go Nebulizer Instruction Manual for more complete cleaning instructions.

Important Cleaning Instructions



To optimize the performance and life expectancy of your Aeroneb Go device, follow these cleaning instructions carefully.

AFTER EVERY USE:

Disconnect the power source before you take the nebulizer apart to rinse it.

1. **Rinse the Nebulizer Unit** (Medication Cup, Nebulizer Body, and Mouthpiece or Mask Adapter and Mask) **under running hot tap water after every use.**
2. Shake off excess water and allow parts to air dry on a clean, dry towel, out of the reach of children.
3. Make certain that all cleaned parts are completely dry before you store or use them.

NOTE: Water remaining on the underside of the Aeroneb[®] Go generator after cleaning may cause the unit not to work. Be sure the unit is completely dry prior to use.

DAILY:

Disconnect the power source before you take the nebulizer apart to clean it.

1. Wash the Nebulizer Unit (Medication Cup and Cap, Nebulizer Body, and Mouthpiece or Mask Adapter and Mask) by hand in a solution of warm water and mild liquid dishwashing soap. Use soap intended only for washing dishes by hand. Do not use dishwasher detergent as it is too abrasive. Do not leave parts soaking in soapy water for more than 20 minutes.
2. Rinse all pieces thoroughly under running hot tap water. Make sure that no soap is left on any of the parts.
3. Shake off excess water and you may use device. Keep out of the reach of children.
4. Make certain that all cleaned parts are completely dry before you store or use them. Do NOT remove the retaining ring under the medication cup.

NOTE: Water remaining on the underside of the Aeroneb[®] Go generator after cleaning may cause the unit not to work. Be sure the unit is completely dry prior to use.

NOTE: Do NOT clean the nebulizer in a dishwasher.

OPTIONAL DISINFECTION:

If you would like to disinfect your Aeroneb[®] Go Nebulizer, you may use the following method.

VINEGAR SOLUTION:

- I. First, clean the Nebulizer Unit as described in the daily cleaning section, following steps 1- 2.
- II. Fill a container with a mixture of three (3) cups hot water to one (1) cup white distilled vinegar. (example: Heinz Distilled White Vinegar)
- III. Soak the Nebulizer Unit parts (Medication Cup and Cap, Nebulizer Body, and Mouthpiece or Mask Adapter and Mask) in the solution for thirty (30) minutes.
- IV. Rinse all pieces thoroughly under running hot tap water.
- V. Shake off excess water, and allow parts to air dry on a clean, dry towel, out of the reach of children.
- VI. Make certain that all disinfected parts are completely dry before you store or use them.