

TranSport ^{evo}

Battery Instructions

The TranSport battery requires some simple precautions to ensure proper operations and to avoid damage to the device or the battery. When used properly, the battery can provide two treatments of about 20 minutes each. Battery operation may vary because of charge conditions and care.

Battery Specifications

Part Number: 5055BAT

Battery Type: NiMH 1500 mAh


Supply Voltage: 15VDC use only original TranSport adapter

Complete Charge: At least 4 hours

Capacity: 1500mAh

To Use and to Charge TranSport Battery

1. Remove the rubber cushion before inserting the battery. Insert the battery by matching the projection on the top of battery and groove on the bottom of the device. Slide it until it clicks to have installation done. Slide it in an opposite direction to remove the battery.
2. Make sure the battery is fully charged before use. Recharge the battery by inserting plug of the AC adapter into the power socket of the device when battery is installed. The suggested charging time is 6-8 hours for the first time. The typical charging period is 4 hrs for daily use. Once the battery is fully charged, it can provide two treatments of about 20 minutes each.
3. When the power in the battery pack becomes low, the orange battery status indicator flashes to indicate that only a few minutes of battery power remains. You must recharge battery pack or use AC power source. During the charging operation, the orange battery status indicator will light and automatically turn off after the battery is fully charged.

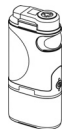

CAUTION - When AC power cord is not connected, the unit's main power source is from the rechargeable Ni-MH battery pack. Note that even when the device is turned off, it is still in the stand-by condition and consuming the battery power. It is suggested that users remove the battery pack when the device is not used for prolonging battery life.

CAUTION - If the charging time is over 24 hrs but the battery charge indicator still lights up, please do NOT continue to charge the battery but disconnect the adapter for safety consideration.

CAUTION - The TranSport battery has protection from overheating and will not operate if temperature is 40°C (104°F) or higher. Allow battery to cool before restarting.

CAUTION - In case of failure or malfunction, turn off the device and read the instruction manual. Do not attempt to open the plastic case or to replace any components. The battery is a whole sealed pack and cannot be repaired.

NOTE - Battery will not charge while device is in use.

NOTE - In order to insure maximum performance of the battery, routinely charge the battery when not in use.

WARRANTY

evo warrants the TranSport rechargeable battery to be free from defective workmanship and materials for a period of 6 months from the date of purchase. This warranty is limited to the dealer. Any defective part or assembly will be repaired or replaced, at the sole discretion and determination of evo if the unit has not been misused or tampered with during the warranty period. Shipping charges, if any, shall be paid by the purchaser.

NOTE: There is no other express warranty. Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to the duration of the express limited warranty and to the extent permitted by law and all implied warranties are excluded. This warranty does not cover providing a loaner compressor, compensating for costs incurred for compressor rental, or labor costs incurred in repairing or replacing defective part(s).

DISTRIBUTED BY:



evo Medical Solutions

2636 289th Place

Adel, IA 50003

515.993.5001

515.993.4172 f

800.759.3038 (toll free)